

USER MANUAL

Steel 4-wheeled rollator

Product code: TGR-R RS 880

Dear Customers,

Thank you for purchasing our product. We hope that the equipment you have selected will meet your expectations. We would like to share some important information with you.

Timago International Group is a Polish company which distributes medical, rehabilitation and orthopaedic products to pharmacies and medical stores.

Our mission is to ensure your comfort by offering safe, functional, and high-quality products.

We have many years of experience in this sector. Timago products reach a wide range of users in Poland and throughout Europe through our distributors (medical retailers).

Timago's company culture promotes openness to your needs. Our goal is to promote safe and correct use of our products for treatment and rehabilitation in order to improve comfort and quality of life.

We invite you to access our website for more information: **timago.com**.

1. Introduction

Prior to use of our products you should inspect the equipment for any defects which may have occurred during transport. If this happens, please contact your retailer.

Then, please read this instruction which includes important information for the user.

Please contact us if you have any questions concerning the use of the equipment. Our advisors will provide you with any information you require on an ongoing basis.

Please note that if you have questions other than those concerning the use of our products, you should contact your doctor, nurse or physical therapist.

2. Characteristics


Folding 4-wheeled rollator is made of steel powder coated frame. The rollator possesses upholstered seat, soft backrest and removable basket. The product is equipped with height adjustable handles with anatomic handgrips and brakes with locking mechanism. The rollator is supplied with wheels with narrow cross section 8" in diameter.

The set includes:

- Two handles with brakes
- Frame with seat
- Backrest
- Basket
- Locking knobs – 2 pcs

Elements of the rollator:

- a. Push handle
- b. Brake
- c. Locking knob
- d. Brake cables
- e. Backrest
- f. Locking pin
- g. Side crossbar
- h. Hinge of side crossbar
- i. Strap for folding rollator
- j. Basket
- k. Upper adjustment of brake
- l. Lower adjustment of brake

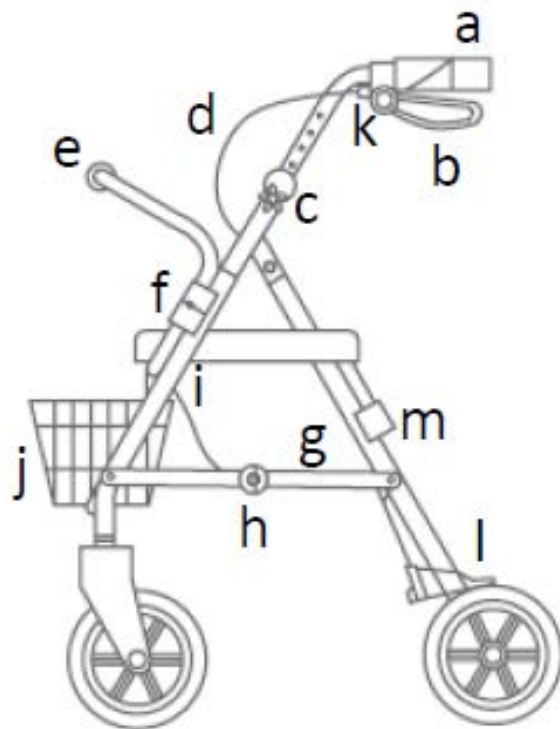

3. Application

The rollator is designed for patients with reduced mobility who need stable support and help while walking. It is also recommended to the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the rollator affects positively on the return to activity of the people with locomotion and tilting to erect position problems. It guarantees better stabilization, support and security while walking.

4. Assembly and general recommendations for use

Note! Follow the information contained in this manual while assembly and adjustment. Users with reduced mobility may need help from a third person who has appropriate skills.

Note! Do not use the rollator if any defects or deficiencies (e.g. cracks) are noticed due to the risk of accident.

Unfolding and folding the rollator

- To unfold the rollator, press the side crossbar, which is placed under the seat, until the hinges are completely unfolded. Then, fix the handles into the mounting openings and select appropriate height by tightening the locking knobs. Brake cables should be placed at the front. Lean the seat against unfolded frame.
- Fix the backrest into the mounting openings at the front of the rollator until the pins are locked.
- The basket can be hanged on the frame.
- Remove plastic casing from wheels.
- To fold the rollator for transport, remove the basket, lower the seat and pull upwards the strap which joins the side crossbar with frame until the frame is completely folded.

NOTE! Be careful while folding or dismantling the rollator due to the risk of smashing fingers in hinges.

Brakes

Note! The assistance of a third person may be required while first use regardless of the user's health condition.

- To stop the rollator, pull the brake handle upwards and hold.
- While stop or using the seat, lock the brake with locking mechanism by pushing down the brake handle.
- To release the brake, pull the brake handle upwards.

Height adjustment of handles

The height of the handles should be adjusted so that the user can stand upright, as far as he or she is able to, with his or her shoulders in rest position and with slightly bent elbows. It is inadvisable to hang on the rollator with all the body weight.

- To adjust the height of the handles, unscrew the locking knobs which are located on the outer side of the frame, select appropriate height and tighten the locking knobs.

Note! The handles must be set on the same level.

Adjustment of brakes

Note! Check the condition of brakes before first use. When the brake handles do not brake equally, they should be adjusted.

Adjustment of upper brake

Unscrew the nut of the cable, pull it up to the appropriate length and lock it by tightening the locking screw.

Adjustment of lower brake

Loosen the mounting screw with the use of open-end wrench, pull the black brake cable to adjust the tension of the cable, and then, tighten the mounting screw.

5. Technical data

Overall width	60 cm
Overall depth	73 cm
Seat dimensions	30 cm x 30 cm
Seat height from the ground	53 cm
Handles height adjustment	81-93 cm
Weight	9.2 kg
Maximum capacity	135 kg
Maximum capacity of the basket	5 kg
Frame colour	dark blue

6. Notes and recommendations

- Only one person can use the rollator.
- The seat can be used only for temporary rest.

- Before using the seat, engage the brake.
- Do not use the seat for transport purposes.
- Do not lean backwards while sitting on the rollator due to the risk of accident.
- Do not stand on the rollator.
- The basket is designed to carry only lightweight and small objects.
- Do not hang any objects on the rollator due to the risk of accident.
- The rollator is designed for users with maximum weight 135 kg.
- The rollator is designed to be used on even surfaces with inclination less than 5%.
- Due to the risk of falling, it is unacceptable to use the rollator on wet, icy, greasy, and strongly sandy surfaces as they do not provide firm support.
- Remove all potential obstacles in the room such as too high carpets or thresholds, and secure slippery floor with anti-sliding mats.
- Avoid wet surfaces and use the rollator when the light is on.
- Be particularly careful when walking on uneven pavement, overcoming curbs. Do not walk backwards.
- Do not make any repairs or modifications by yourself. Any repairs should be done by qualified personnel of the manufacturer.
- It is essential to maintain all the elements of the rollator clean and to store the rollator in an appropriate way. Storage in rooms with very high humidity (baths, laundries, etc.) or in an open air in combination with lack of hygiene will result in a rapid loss of usable and aesthetic parameters.

NOTE! Timago International Group does not bear responsibility of improper use of the rollator, failure to comply safety regulations and misuse.

7. Cleaning and maintenance

Reviews

Make visual inspection of all the joining elements (clamps, hinges) of the rollator at least once every three months including:

- Brakes
- Mounting elements (pins, screws, knobs, nuts)
- Height adjustment of handles
- Stability and look of the frame

Cleaning

- Clean the frame with damp cloth and dry thoroughly.
- Do not use strong polishing agents due to the damage of protecting coatings.
- Clean the dirt from moving parts of the rollator regularly (hinges, clamping screws, locks).

8. Warranty information

All products distributed by our company are under a warranty, the terms of which are described in the warranty card received at the time of purchase.

Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

As a user friendly company, we only supply products which have been thoroughly tested with respect to materials, quality and functionality.

Please contact us if you have any questions concerning our servicing procedure.

9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date
	Serial number		Representative in European Union
	Note		Please read the instruction
	The Manufacturer checked for compliance with essential medical devices requirements.		

10. Recommended products

- Products offered by Timago International Group include:
- Orthoses and joint stabilizers
- Manual and special-use wheelchairs
- Walkers and rollators
- Crutches and canes
- Rehabilitation beds
- Ancillary and sanitary equipment
- Anti-decubitus equipment and accessories

- Stabilizing and immobilizing belts
- Electro-medical equipment
- Disposable medical equipment

Our complete line of products is available from our distributors.

Learn more at: timago.com.



Manufacturer:

Timago International Group

Spółka z o.o. i Spółka – Spółka komandytowa

ul. Karpacka 24/12

43-316 Bielsko-Biała, Polska

T.: +48 33 499 50 00

F.: +48 33 499 50 11

E.: info@timago.com



Issue II, date 22.10.2018